


How to Decommission an EauFlow RO water filter for an extended period of inactivity (1 week+). 30 minutes / Easy


Tick when task complete

- 1 Turn off mains water coming into RO
- 2 Drain tank (by opening tap)
- 3 Close storage tank valve
EF404, Purity, EF505 models = twist blue valve, on top of storage tank, 90 degrees (so it is perpendicular to the tubing).
Clarity/EF189 = twist blue valve clockwise (blue valve can be found in top right corner at the back)
- 4 Turn off electricity
- 5 Remove all 5 cartridges and dispose of in general waste*
- 6 **Hard water system only:** remove large blue cartridge (EY30810)
Method: Pinch the quick-connection collar where tube connects onto cartridge, between your index and middle finger of one hand and pulling tube away with other. You return to CleanCert for recycling in protective packaging.

*How to remove cartridges

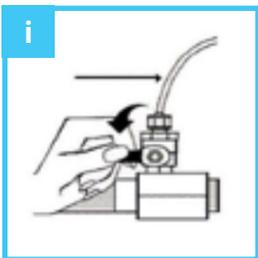

Turn off mains water

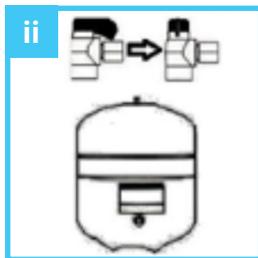


Close



Open the door



Pull up to take off cover

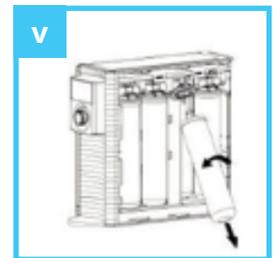

Remove used filter

31/3/20. v1. Decommission RO.

Please keep for your records

Carried out by (name):

Date: