

3 MONTHLY

Monitoring 'Organic' levels - (bacteria)

We recommend using a TVC dipslide to proactively monitor bacteria in your RO unit.

Instructions

- Allow RO water to run for 30 seconds.
- Remove cap & insert dipslide in water for 10 seconds
- Incubate for 1 week to get an accurate reading.
- Repeat quarterly.

Date	Test Carried Out	Dipslide Batch No	Result	Action To Follow?

If you have a positive reading on your dipslide:

1. Empty RO tank completely and allow to fill up. Then retake dipslide test.
2. Ensure the water feeding your RO system is not coming from a cold water storage tank in the loft, or has any plumbing 'dead legs' on it.
3. Visit our website (www.cleancert.co.uk) for a Biofilm Cleaning Kit (Ref: BFCDS01) and CleanCert Biofilm Disinfectant (5L), which should resolve your issue. Instructions for this process can be found on: YouTube type in 'Clean CleanCert RO' and select your model.

If you have any further questions please contact the technical team on:
08443 511115 or email technical@cleancert.co.uk