


Tetric® plus


1

Self-Etch

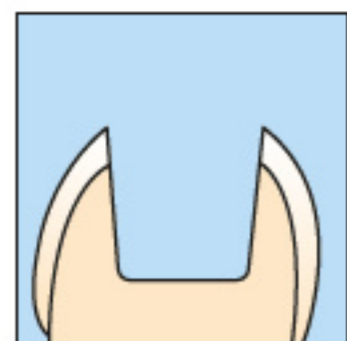


or

Etch & Rinse

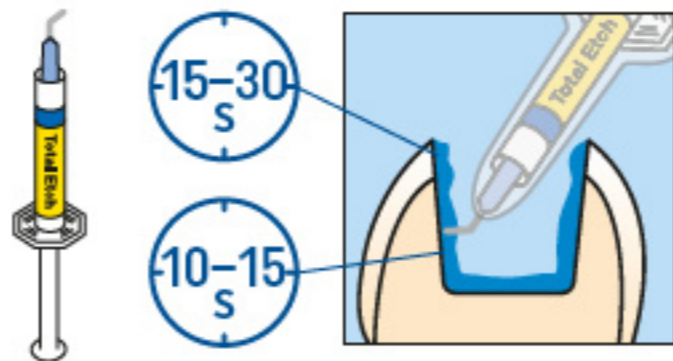





or

Selective Enamel-Etch

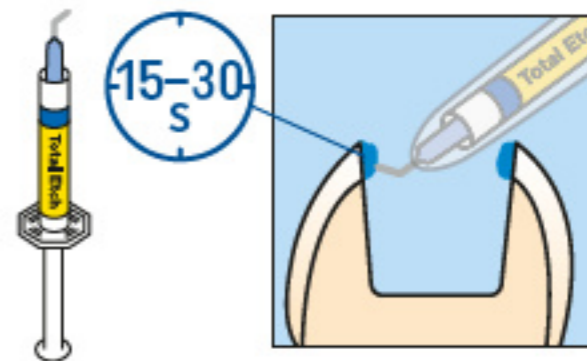


2

Adhese® Universal
VivaPen®

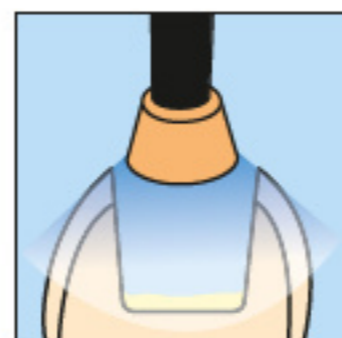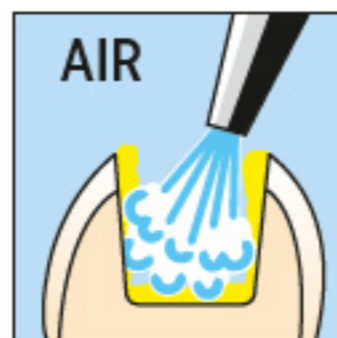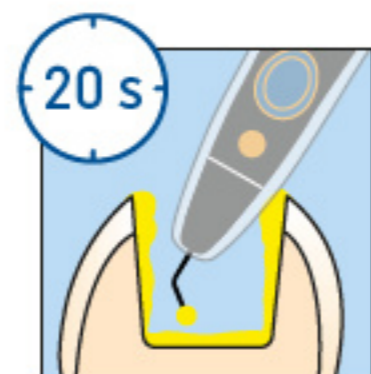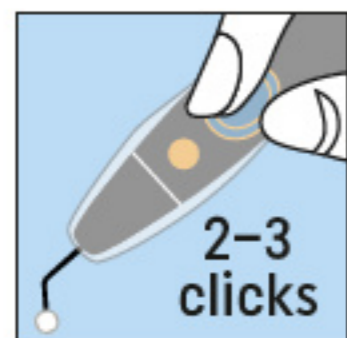


3

Tetric® plus
Flow



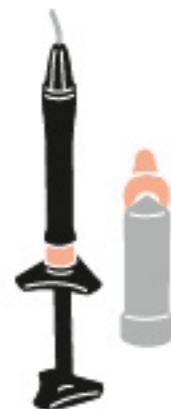


As initial layer

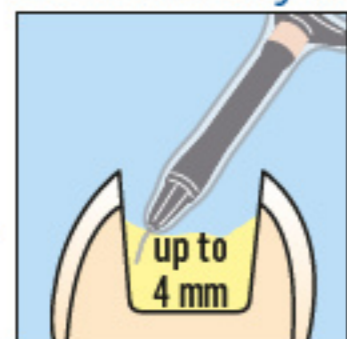




or

Small restorations without a capping layer

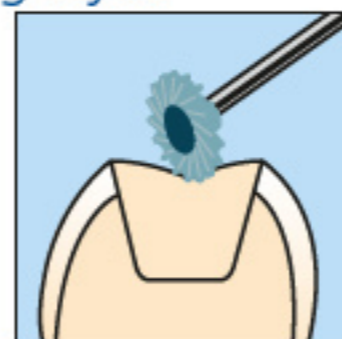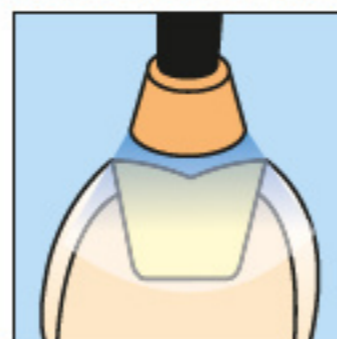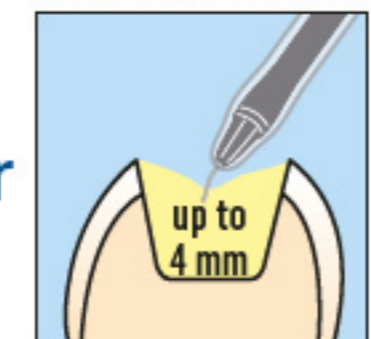

4

Tetric® plus
Fill

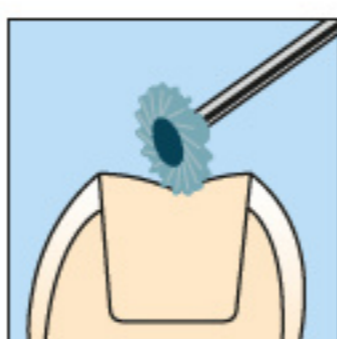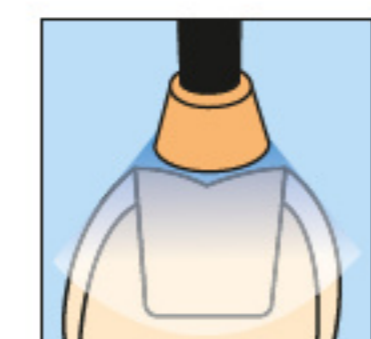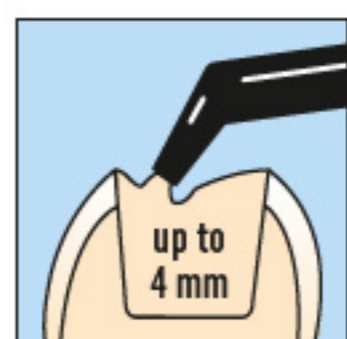

Bluephase®
PowerCure



For permanent teeth only

For deciduous teeth see instructions for use

	Tetric plus
500 – 900 mW/cm ²	20 s
1,000 – 1,300 mW/cm ²	10 s
1,800 – 2,200 mW/cm ²	5 s
2,700 – 3,300 mW/cm ²	3 s

Date information prepared: 2023-10-12
762305 / Rev. 0

ivoclar

CE 0123

see
Instructions
www.ivoclar.com/eIFU

Rx ONLY
For dental use only!